

Audicus

Instruction Manual

Volume Remote

Thank you for choosing Audicus!

We are excited for you to experience our hearing aids.

Your Volume Remote will enhance the way you use your hearing aid.

If you have trouble using your remote, feel free to contact us at 888-979-6918 or help@audicus.com.



Battery Information

Battery size



AAA

Low battery warning




Light indicator
changes to yellow
or red

Changing batteries



Volume Remote Overview

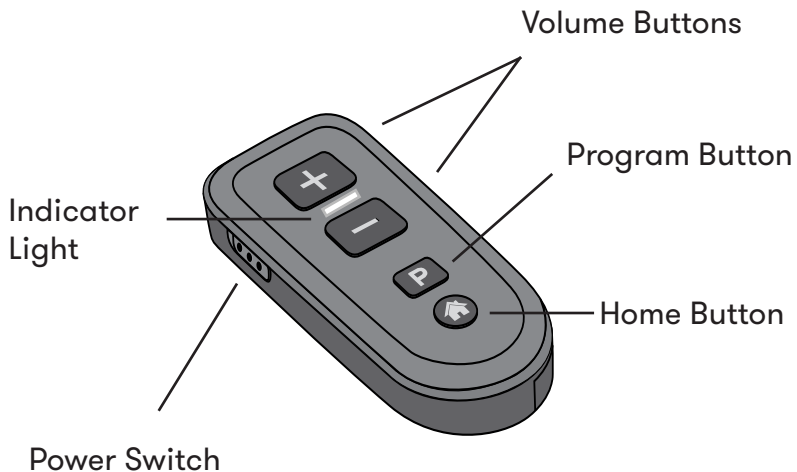

Features Overview

Your hearing aid(s) has been paired with the remote and functions as outlined in this manual.

Feature	Function
Power Switch	turns remote on or off
Volume Buttons	increase or decrease volume
Program Button	cycle through programs upward (1 to 5)
Home Button	default to program 1
Light Indicator	provides status of remote

NEVER connect the remote to USB cable. The USB port is used with a special software to couple it with the hearing system.

Getting Started

Turn on your remote by sliding the switch up.

To turn off, slide the switch down.



LED Light	Status
Green	battery OK
Red	Low battery (<10%)
2 second blink (green)	Ready to use
No light after pressing button	Dead battery

Programs/ Volume

Changing Programs

The Alto has two programs: Smart Program and Loop Program.

The Smart Program is an auto-adaptive setting in which the hearing aid detects your environment and provides appropriate amplification. The Loop program lets you use telecoil in places like auditoriums or theaters. You will hear one beep for the Smart Program and three beeps for the Loop Program.

Press the **P button** to change programs.

If you put a phone close to your ear, you will hear two beeps. This is a phone mode that activates automatically and enhances speech for the phone.

Changing Volume

Press the "+" or "-" button. When you have maxed out the volume, you will hear a series of beeps. There are 5 louder steps and 5 softer steps from default.

Troubleshooting

Issue	Cause
Light doesn't work	Remote is off
	Dead battery
	Battery inserted incorrectly
Hearing aids do not respond	Remote is too far away
	Hearing aids are dead

Considerations

Keep remote away from pets and children under 3.

Remote control uses low power signal to communicate with hearing aids.
Please KEEP away from breast pocket if you use a pacemaker.
Remote might interact with pacemaker.

Dispose of batteries appropriately.

Only use with hearing aids programmed for you.
Do not alter the remote without approval.
Do not use the remote in explosive areas or where electronic equipment is prohibited.

Keep remote dry at all times. Do not expose to water. If wet, let dry naturally by uncovering the battery door and removing battery.

Use high quality, AAA alkaline batteries. Do not use rechargeable batteries.

Keep remote at least 10cm away from magnets.

